










Laboratory Collection Guide for Non-blood Specimens

Refer to our electronic test catalog for our full test menu and complete collection instructions.

<https://unitypointfinley.testcatalog.org>

Test	Collection Device Name	Special Notes
<p>Bacterial Cultures (aerobe and anaerobe)</p> <p>Fungal Cultures</p> <p>Strep A PCR</p>	<p>ESwab</p> 	<p>The caps may be white, blue, or green depending on the swab size.</p> <p>Write the source on the patient label. Example: Left leg wound.</p>
<p>Chlamydia</p> <p>Gonorrhea</p> <p>Trichomonas</p> <p>Vaginitis Molecular testing</p>	<p>GeneXpert Collection Swab</p> 	<p>Use for female genital specimens, male/female rectal specimens, and male/female throat specimens.</p> <p>For male patients: chlamydia, gonorrhea, and trichomonas is not available on genital specimens. Submit a urine specimen for male patients.</p>

<p>COVID</p> <p>Influenza A & B</p> <p>RSV</p> <p>Herpes Simplex 1 & 2</p> <p>Varicella Zoster</p> <p>Mumps</p>	<p>Viral Transport Media (multiple vendors available)</p> 	<p>Also may be called UTM, VTM, or M4 viral media.</p> <p>Write the source on the patient label.</p>
<p>MRSA Nasal Screening</p> <p>MRSA detection from soft tissue wounds</p>		<p>Write the source on the patient label.</p> <p>The soft tissue detection is a molecular (PCR) method and does not replace culture.</p>
<p>Stool Culture</p> <p>GI Filmarray Panel (stool)</p>	<p>Cary Blair Media (Green Cap)</p> 	<p>Fill to the indicator line on the vial.</p> <p>Complete and submit the patient history form with the specimen.</p>

Ova and Parasites (stool)	<p>SAF Media (Black Cap)</p> 	<p>Fill to the indicator line on the vial.</p> <p>Complete and submit the patient history form with the specimen.</p>
<p>C diff toxin</p> <p>Fecal leukocytes</p> <p>H. pylori Antigen</p> <p>Norovirus</p>	<p>Sterile Cup</p> 	<p>Refrigerate promptly.</p> <p>For C diff toxin: formed stool is not acceptable and will be rejected.</p>
Fecal Occult Blood	<p>iFOB Collection Kit</p> 	
Pinworm	<p>Pinworm Paddle</p> 	<p>Do not use Scotch tape for collection. Specimen will be rejected.</p>

